

	Computer Sports Medicine, Inc.			
	Title:	Printing the CYBEX 6000 Error Log		
	Document #:			
	Rev:	A Draft	Effective Date:	
	DCO #:			
	Page 1 of 1			

The Critical Error Log is part of the CYBEX 6000's built in diagnostics. It is often useful in pinpointing the source of problems with the CYBEX 6000. This document gives procedures for viewing the Critical Error Log and for sending a copy to CSMI.

FAXing the Log to CSMI

1. At the CYBEX System Menu, select **System Master Utilities, Error Management, View Critical Error Log.**
2. Press **F10** to print the log.
3. FAX the three most recent pages of the log to CSMI at 781-297-2039. Please include your Name and Phone Number on the FAX.

E-mailing the Log to CSMI

1. Access the **DOS** prompt on the CYBEX 6000 computer.
2. Copy the file, **C:\CYBFILE\CRITERR.LOG** to a floppy disk
3. From a computer with Internet access, sent an email to info@csmisolutions.com the **CRITERR.LOG** file as an attachment. Please include a description of the problem you are seeing with the CYBEX machine in the text of the email.