

Determining your Biodex Revision

1. Introduction

- 1.1. Biodex produced a number of Systems and Revisions. This document provides instructions for identifying your Biodex System and Revision.

2. Early Systems – Not HUMAC Compatible.

- 2.1. A dual-chair system with a dynamometer permanently mounted between the chairs.
- 2.2. The front panel is Black and does not say “SYSTEM 2” on it.

3. System 2 – HUMAC Compatible.

- 3.1. A dual-chair system with a dynamometer permanently mounted between the chairs.
- 3.2. The front panel is white says “SYSTEM 2” on it.



4. System 3, Rev 1 – HUMAC Compatible.

- 4.1. A single chair system.
- 4.2. Rev 1 has a round plastic DYNAMOMETER SIGNAL cable.



Figure 1 Rev 1 Dynamometer Signal Cable

5. System 3, Rev 2 – Not HUMAC Compatible.

5.1. A single char system.

5.2. Rev 2 has a rectangular metal DYNAMOMETER SIGNAL cable.



Figure 2 Rev 2 Dynamometer Signal Cable

5.3. Rev 2 systems started with Serial Number 07021316.

6. System 4 – Not HUMAC Compatible.

6.1. A single char system.

6.2. A rectangular metal DYNAMOMETER SIGNAL cable.