



CSMi CONFIDENTIAL

101 Tosca Drive
Stoughton, MA 02072 USA
(Phone) 781.297-2034
(FAX) 781.297-2039
(Web) www.csmisolutions.com

Sales Order Number: _____

Dear Customer

This document confirms your HUMAC NORM installation is complete and in good working order. Please carefully review the following list. If you notice any discrepancies note them on this form. Your CSMi representative will leave the original copy of this document.

Contact Name: _____

Clinic Name: _____

Street (1): _____

(2): _____

City/State/Zip: _____

Country _____

Phone: _____

e-mail: _____

HUMAC S/N: _____

Confirmation Installation is Complete and System is Accepted by Customer See Checklist on Page 2	
CUSTOMER SIGNATURE / DATE	CSMi Representative SIGNATURE / DATE
Printed Name	Printed Name

Instructions

1. FAX this, the Mock Test Report and the Software Registration form to CSMi at 781-297-2039.
2. Leave original copies with the customer.

HUMAC2015 NORM Upgrade Installation Steps

Initial the completed column as each step is performed.

Completed	Doc Nbr	Instructions
		Start the Original CYBEX application. Confirm the NORM runs correctly in Isokinetic mode. Ask the customer if there are any known issues with the NORM. If there are any issues, indicate them in the Comments section below and contact CSMi before proceeding.
		Record the HUMAC Serial Number on Page 1 of this form.
	300910	Setting up a HUMAC2015 Computer
	300025	Disabling the USB Power-Down
	300604	Setting the HUMAC Windows Power Settings
	300911	Setting the HUMAC2015 Printer Margins
	300904	HUMAC2015 Software Installation Instructions
	300917	HUMAC2015 NORM Interface Installation Instructions
	300914	Verifying the HUMAC2015 Servo Settings
	300912	Registering the HUMAC2015 Program
	300916	Recording the HUMAC2015 NORM Calibration Weight Values
	300906	Calibrate the NORM
	300915	Performing a HUMAC2015 Mock Test
		Press the comfort switch and confirm Error 2001 is reported.
		Perform the clinical in-service

Notes:

¹ These steps have been performed at the factory for computers supplied by CSMi and need not be repeated during the installation.

Comments: