



Setting the HUMAC2015 Printer Margins

1. Introduction

1.1. This document defines the steps for setting the computer to print HUMAC reports.


2. Procedure

2.1. Start the **Internet Explorer**. If Windows attempts to connect to the Internet or starts the Internet setup wizard, click the **Cancel** or **Work Offline** buttons until the Internet Explorer starts.

2.2. From the Internet Explorer **File** menu, select **Page Setup**.

2.3. Clear the **Header** and **Footer** values.


2.4. English Units

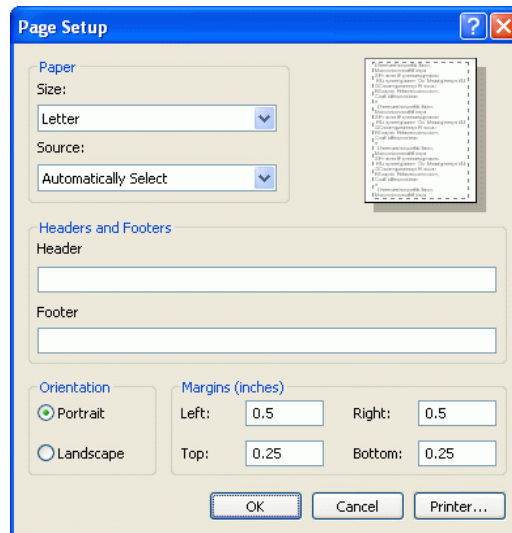

2.4.1. Set the **Left** and **Right** margins to **0.5** inches.

2.4.2. Set the **Top** and **Bottom** margins to **0.25** inches.

2.5. Metric Units

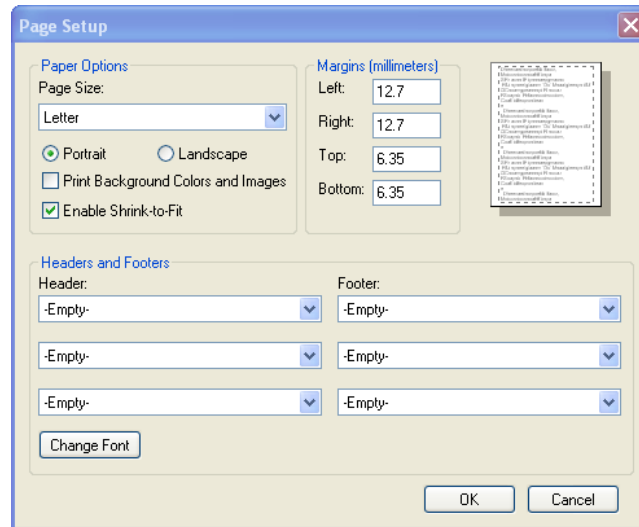

- 2.5.1. Set the **Left** and **Right** margins to **12.7** millimeters.
- 2.5.2. Set the **Top** and **Bottom** margins to **6.35** millimeters.
- 2.6. Click the **OK** button to save the settings.