


Registering the HUMAC2015 Program

1. Introduction

- 1.1. The HUMAC program must be registered with CSMi. The program will run for 45 days before registration is mandatory.

2. Procedure

- 2.1. Start the HUMAC program.
- 2.2. If the HUMAC has not been registered, the program will display the registration screen. This screen shows the number of days remaining in the trial period. Once the trial period expires, the program will only run in Demo mode without the registration code.

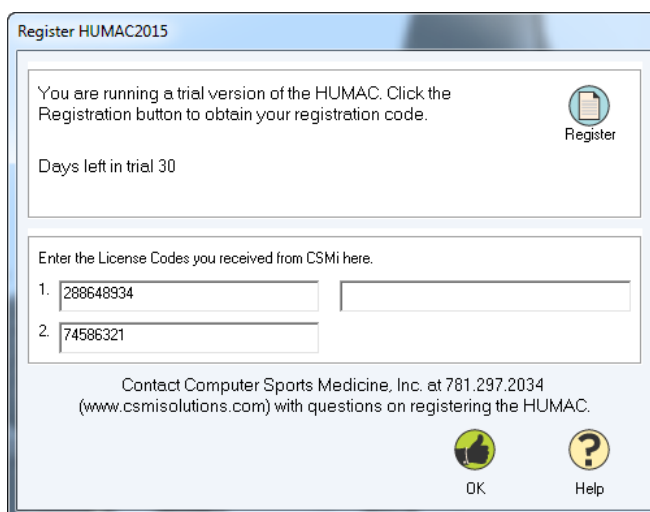

- 2.3. **Note:** If you are not ready to enter the registration code, click the **OK** button to continue with the HUMAC program during the trial period.

3. Registering by the Internet

- 3.1. Go to www.csmisolutions.com.
- 3.2. From the **Service and Support** menu, select the **Software Activation** link.
- 3.3. Enter the two codes you see on the HUMAC screen separated by a /.

<p>Enter Activation Code</p> <input type="text"/> <p><small>(Eight Digit Code on Left side of License screen in this format xxxxx-xxxx)</small></p>	<p>Enter Activation Code 2</p> <input type="text" value="5986445"/> <input type="text" value="15497856"/> <p><small>(Eight Digit Codes on Left side of License)</small></p>
<p>Product Requiring Activation? (required)</p> <p> <input checked="" type="checkbox"/> HUMAC for NORM <input type="checkbox"/> HUMAC for Biodex <input type="checkbox"/> HUMAC Balance <input type="checkbox"/> HUMAC 360 <input type="checkbox"/> Group Test <input type="checkbox"/> Research Toolkit <input type="checkbox"/> SportsWare </p>	
<p>Software Version Activating (required)</p> <p> <input type="radio"/> HUMAC 2004 <input type="radio"/> HUMAC 2009 <input type="radio"/> HUMAC 2014 <input checked="" type="radio"/> HUMAC 2015 </p>	

- 3.4. Complete the form, including the **Product Requiring Activation** and the **Software Version Activating**.
- 3.5. Click the **Submit** button.
- 3.6. CSMi will e-mail your activation code.
- 3.7. **Note:** You can click the **OK** button to continue with the HUMAC program during the demo period and then enter the code you receive from CSMi later.

4. Registering by FAX

- 4.1. Start the HUMAC program. The **Register HUMAC2015** form will be displayed.
- 4.2. Click the **Register** button.
- 4.3. The **Registration** screen will be displayed.

Register HUMAC2015

It is important that you register the new version of your HUMAC program with CSMi. Registering your program allows us to keep you informed of important changes and new features in future versions of the HUMAC program.

NORM Serial Number

HUMAC Serial Number

Your Name

Clinic Name

Address

City, State, Zip

Country

Phone

FAX

e-Mail

Send me e-mails announcing HUMAC updates.

The information in these fields is required.

- 4.4. Enter the information and click the **Print** button.
- 4.5. FAX the printed form to CSMi. The FAX number is located on the form.
- 4.6. Click the **Exit** button to return to the **Register HUMAC2015** form.
- 4.7. Click the **OK** button to continue with the HUMAC program.
- 4.8. When you receive the registration number from CSMi, start the HUMAC program.
- 4.9. Enter the registration number in the blank space on the **Register HUMAC2015** form.
- 4.10. Click the **OK** button to continue with the HUMAC program.