



Recording the HUMAC2015 NORM Calibration Weight Values

1. Introduction

- 1.1. The NORM ships with four (4) 25 pound calibration weights.
- 1.2. The original NORM weights were manufactured to 25 lbs. The original CYBEX weights have the word "CYBEX" in raised letters on the side of each plate.
- 1.3. The newer CSMi weights are weighed and their weights recorded directly on the weight. The CSMi weights do not have the word "CYBEX" in raised letters on the side of each plate.

2. Procedure

- 2.1. Start the HUMAC program.
- 2.2. From the **Utilities** menu, select **Calibration**.
- 2.3. From the **Calibration** form, click the **Weights** button.
- 2.4. Original CYBEX Weights
 - 2.4.1. From the **Calibration Weights** form, enter the 25 for each **Weight** value and click the **OK** button.
- 2.5. CSMi Weights
 - 2.5.1. From the **Calibration Weights** form, enter the value written on the side of each weight in the corresponding field and click the **OK** button.
- 2.6. Click the **OK** button to close the **Calibration** form.